
AFH Aktiv Tape Exclusive

AFH Rayon Tape Exclusive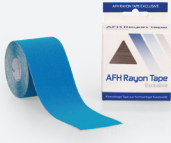

AFH L-Tape Exclusive

AFH Cross Tape Exclusive

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DE

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EN

General application instructions and material properties:

Various materials, shapes, sizes and colors provide optimal application versatility. The tapes are self-adhesive tapes on a high quality cotton base, elastic and air permeable. Rayon tape is made of high quality artificial silk (this increases comfort and water resistance) and is still elastic and air permeable.

Due to the structure of the tape, skin breathing and moisture transport of the skin is not much hindered and remains in balance. Due to the water-resistant acrylic coating, the tapes can be well applied to almost any part of the body. Even when used for several days, the adhesive effect of the tape is maintained. (Exception: when swimming or in similar situations where the body part is completely immersed in water or surrounded by water for a long time). The tape does not hinder the freedom of movement. The tape is latex-free.

Tape adaptation should be performed only by trained therapists or doctors. In case of itching, reddening of the skin or an allergic reaction, remove the tape immediately. Never use the tape on open wounds or as a bandage. No liability is assumed for errors and mistakes as well as any resulting damage.

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