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Abbildung 2

Package Contents: (Figure A)

- Special-Extractor-Pump
- 4 Cupping heads in sizes:
 - ca. Ø 2,5 cm
 - ca. Ø 2,0 cm
 - ca. Ø 1,8 cm
 - ca. Ø 1,5 cm
- Set-Box (plastic)

Figure A

Inserting and removing the various cupping heads: (Figure B)

Screw the special extractor pump vertically into the selected cupping head with a slight turning movement until it is fixed. To loosen the cupping head, simply unscrew the special extractor pump from the cupping head and put the cupping head back in the set box.

Figure B

Product application:

In order to achieve the best possible suction power (the maximum suction power is 750 millibars), place the special AFH extractor pump with the selected cupping head as vertically as possible with a light hand pressure on the desired area and hold it with one hand (Figure 1).

Then use the thumb of the same hand to firmly press down on the AFH extractor plunger (Figure 2).

Please also place the selected cupping head vertically and only with moderate pressure on the special AFH extractor pump so that the cupping head holder of the special extractor pump is not damaged!

Please make sure that you only operate the special AFH extractor pump with one hand and only hold it by the grips provided. This is the only way to achieve optimal and metered suction power!

Figure 1

Figure 2

Caution! Special care instructions for maintaining the long-term quality of your mini cupping set!

Care:

Please do not wash the special AFH extractor pump or let it get wet, as this could destroy the vacuum mechanism inside. You can wash the cupping heads with water or soapy water after use or sterilize them with alcohol. Please do not boil the cupping heads in water, this would damage the contour.

If the special AFH extractor pump is used very frequently, the extractor piston may become difficult to move in individual cases. If necessary, you can improve the gliding effect of the extraction piston only with white medical Vaseline. Then, with the plunger pressed down, carefully pour a small amount of white medical Vaseline through the lower round opening of the special AFH extractor pump (point of application of the cupping heads) for distribution on the inner sliding ring of the vacuum mechanism.

It is important that the suction channel of the extractor piston (visible opening/hole in the middle) does not come into contact with the Vaseline and always remains completely open. Then please operate the AFH extractor piston several times without the cupping head so that the Vaseline is evenly distributed. Operation should then be much easier again.

No liability is assumed for improper handling of the product.